



Bloodlines & substitution fluid systems

For the complete treatment range



AV-06 series




AV-06
single-needle



Online HDF line
C07J-P



Online HDF set
EFL-015

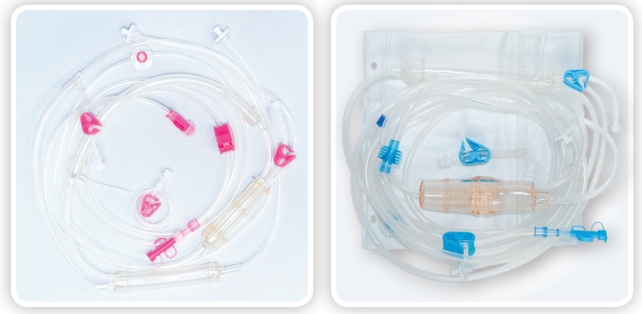

NIKKISO bloodlines convenient & safe

For NIKKISO machines

NIKKISO bloodlines


The AV-06 bloodline series has been specially designed for the use on NIKKISO hemodialysis systems DBB-05, DBB-06 and DBB-07.

Thanks to the universal connection options (post- and pre-dilution), only one bloodline system is required for hemodialysis, hemofiltration and hemodiafiltration. Thus, storage requirements are minimized considerably.



| Bloodlines AV-06 series | | | | |
|---|---------------------------|---|---------------------------------------|-----------------------------|
| Model | Priming volume blood (ml) | Treatment options | Sterilization method | Available for |
| AV-06JA-P with pre-art. drip chamber | 176 | HD, HF, HDF (pre / post) Double-needle dialysis | Steam sterilization (autoclave) | DBB-05 DBB-06 DBB-07 |
| AV-06JC-P without pre-art. drip chamber | 165 | | | DBB-05* DBB-06 DBB-07 |
| AV-06JH-P** with pre-art. drip chamber, saline line and spike | 174 | | | DBB-05 DBB-06 DBB-07 |
| AV-06JA-SN-P | 230 | Single-needle dialysis | | DBB-05 DBB-07 |
| Substitution fluid systems | | | | |
| EFL-015 | | Online HDF set | ETO | DBB-05 DBB-07 |
| C07J-P | | Online HDF line | Steam sterilization (autoclave) | DBB-07 |


* from serial number 68140-01 or 69013-N1
** available on request

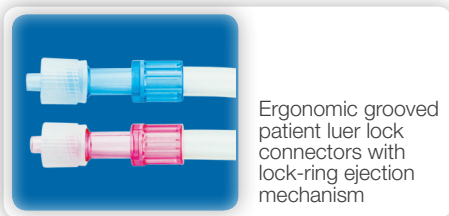

Ergonomic grooved patient luer lock connectors with lock-ring ejection mechanism



Easy to use dialyzer connectors

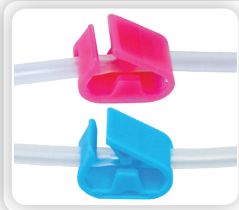


Easy to handle color-coded clamps for arterial and venous line



Large open pressure transducer protector filters with inspection window



Heparin line with smart ergonomic clamp and secured protection cap

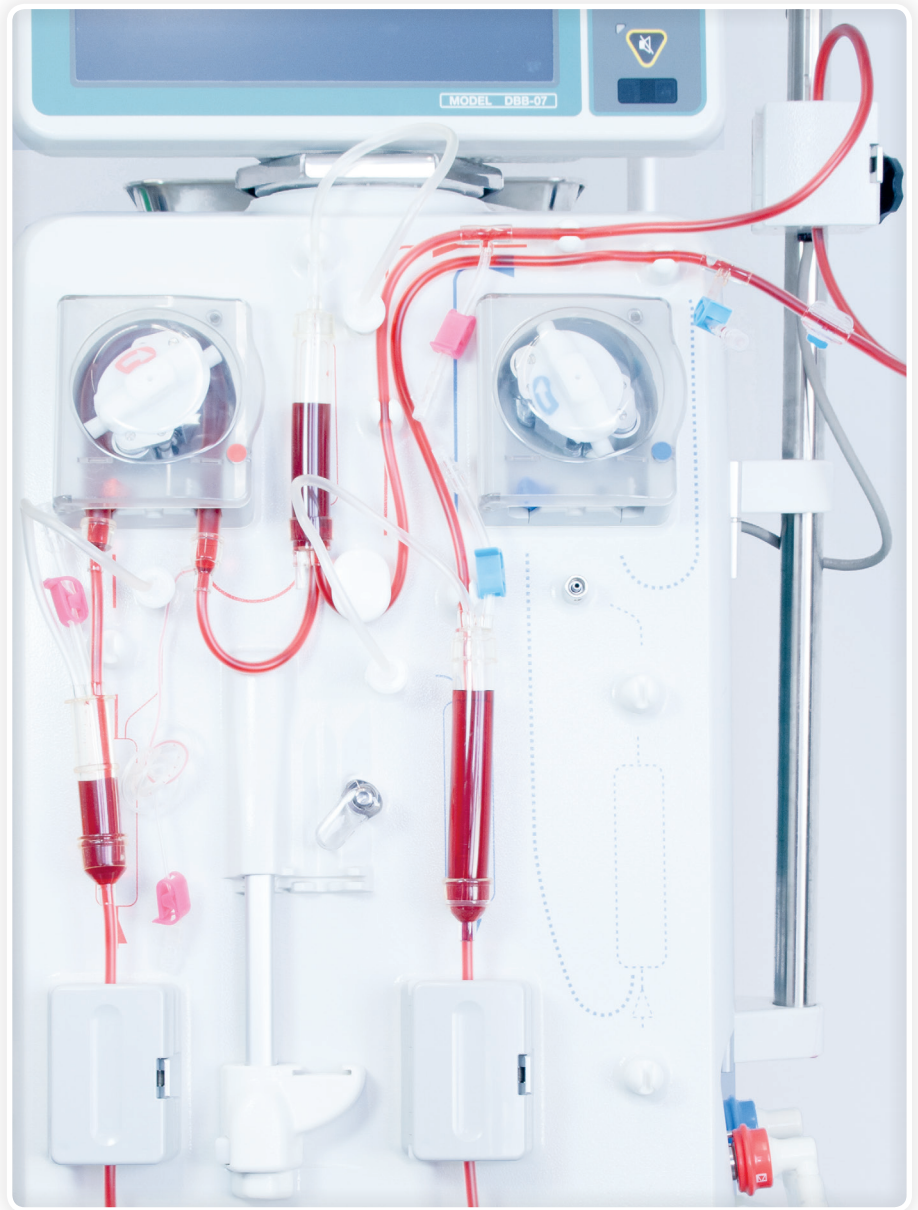

DBB-07 with Haemo-Master and bloodline AV-06JA-P

AV-06 series

The highly efficient steam sterilization method (autoclave) used for the AV-06 series is particularly eco-friendly. The components and drip chambers are designed for a low priming volume and reduced air contact. The recirculation connector (for A/V line) is included as standard. The capless transducer protectors and the perfectly fitting line lengths facilitate the set-up on the machine.

The double-needle bloodline **AV-06JA-P** includes an arterial drip chamber before the blood pump. Furthermore, it has capless transducer protectors and two discharge hooks.

The double-needle bloodline **AV-06JC-P** does not include an arterial drip chamber before the blood pump.

The double-needle bloodline **AV-06JH-P** has an integrated saline infusion line with spike connector.

The single-needle bloodline **AV-06JA-SN-P** is designed for high stroke volumes to meet the best effectiveness of unipuncture treatments.



Treatments

HD, HF, HDF (pre/post), Double-needle dialysis, single-needle dialysis

Sterilization method


Steam sterilization (EN ISO 17665-1:2006)

| | |
|------------------------|---------------------|
| <i>Shelf life</i> | 36 months |
| <i>Packaging units</i> | 24 pieces / box |
| | 528 pieces / pallet |
| <i>AV-06JA-SN-P</i> | 18 pieces / box |
| | 396 pieces / pallet |


Substitution fluid system C07J-P for DBB-07

The integrated cascade filtration with two dialysis fluid filters EF-02D provides the highest quality of dialysis fluid for all treatment types. This guarantees protection to the patient against bacteria, endotoxins and their fragments.

The cost-effective solution for improving patients' quality of life.

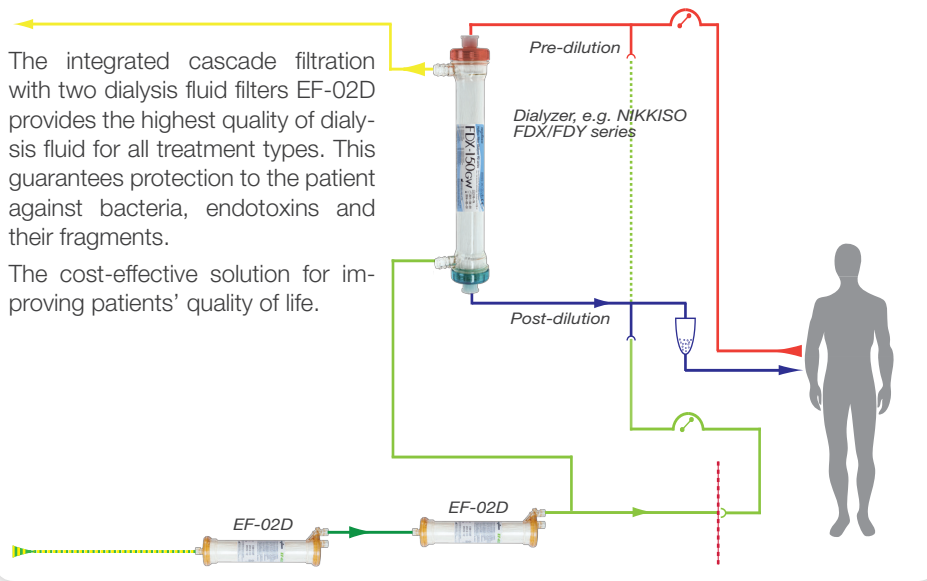


Economic reusable cascade filtration

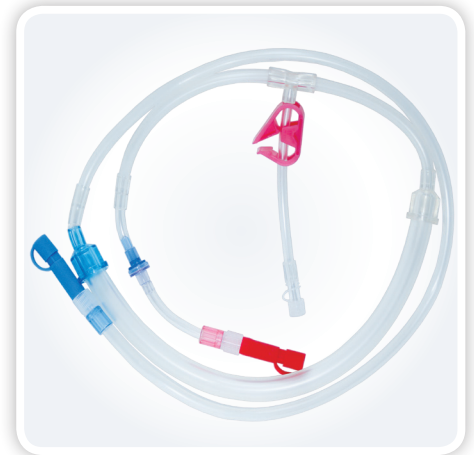

Cost-effective steam sterilized substitution line C07J-P

Substitution fluid system EFL-015 for DBB-05

The additional single use filter in the substitution fluid system EFL-015 provides the last barrier to bacteria, endotoxins and their fragments fulfilling the required double stage filtration for online treatments.

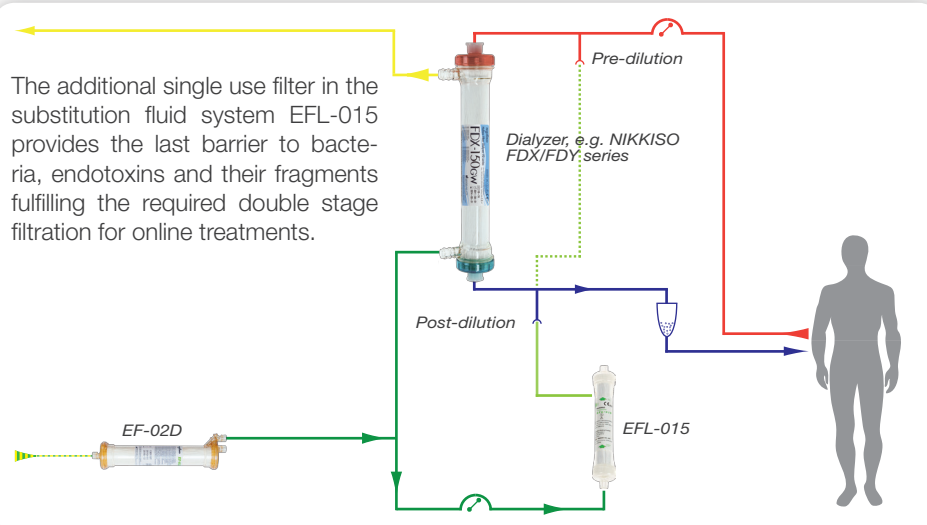

DBB-05 protection system for patients in Online HDF mode



Substitution fluid system EFL-015 including a single use filter

www.nikkiso-europe.eu



Manufacturer
NIKKISO CO., LTD.
20-3, Ebisu 4-Chome, Shibuya-ku
Tokyo 150-6022, Japan

Telephone: +81-3-3443-3727
Fax: +81-3-3440-0681

Website: www.nikkiso.com

European representative
NIKKISO Europe GmbH
Desbrocksriede 1
D-30855 Langenhagen

Telephone: +49 511 679999 - 0
Fax: +49 511 679999 - 11

E-Mail: info@nikkiso-europe.eu
Website: www.nikkiso-europe.eu

Sales office
NIKKISO Europe GmbH
Kapstadtring 7
D-22297 Hamburg

Telephone: +49 40 414629 - 0
Fax: +49 40 414629 - 49

E-Mail: info@nikkiso-europe.eu
Website: www.nikkiso-europe.eu